

## Taking a PD with the digital PD ruler

- 1.) Sit squarely across from patient, just like when taking a PD with a pupilometer or PD stick.
- 2.) Set digital pd ruler on patients face resting on nose like frame.
- 3.) Have person focus at near; align white line at center of each pupil. This is your near PD.
  - To get mono PD measurement – Press R button (hold firmly until measurement stabilizes) for right mono measurement at near and push L button for the Left mono measurement at near.
- 4.) Have patient focus at distance – repeat alignment of white lines for distance PD.
  - To get mono PD measurement – Press R button for right mono measurement at distance and push L button for the Left mono measurement at distance.