



**AMCON
FRAME WARMER
USER'S MANUAL
EQ-3015**

AMCON The Eyecare Supply
CENTER®

1-800-255-6161 • Fax 1-800-397-0013


www.amconlabs.com

AMCON The Eyecare Supply
CENTER®

1-800-255-6161 • Fax 1-800-397-0013

www.amconlabs.com

Thank you for purchasing model EQ-3015 Frame Warmer. It features quick even heating and a strong air output. Please read this manual carefully for convenience and efficiency before your using.

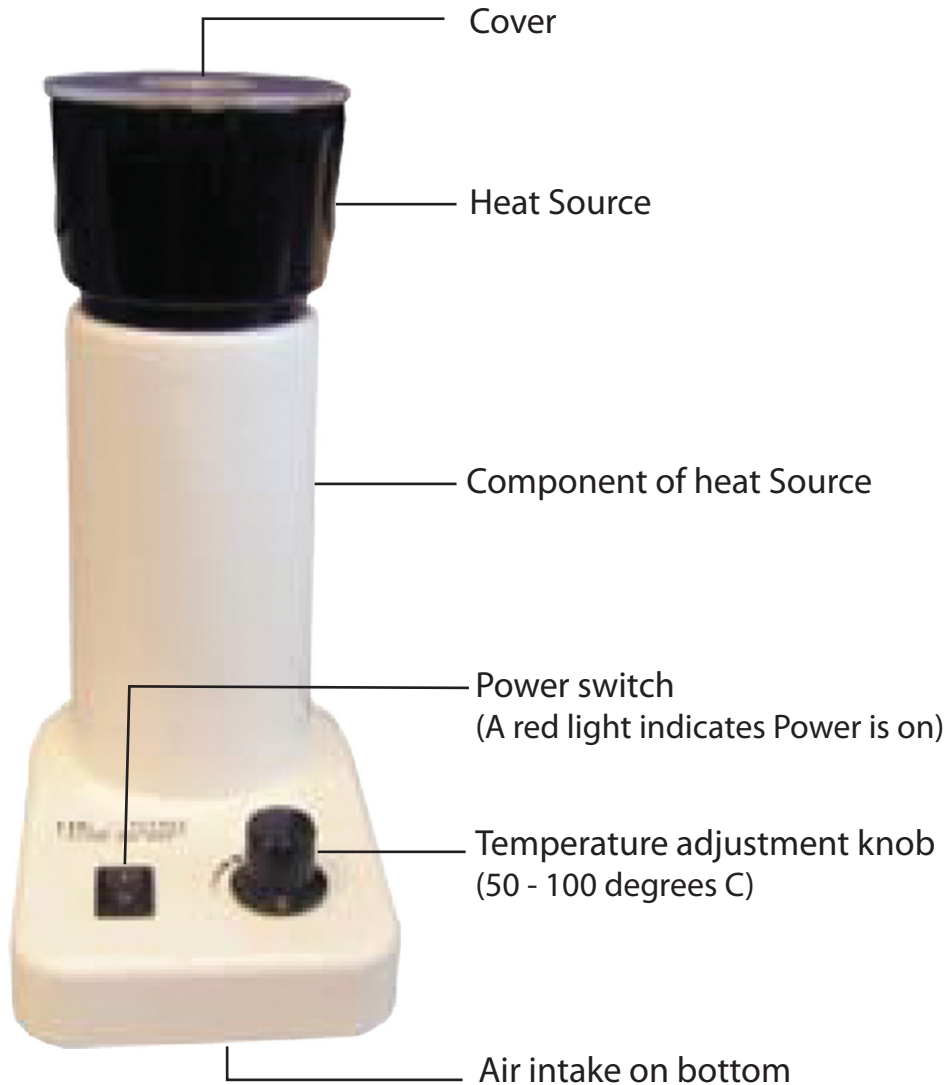

Notice

Before using, please check to see if electrical wires are securely attached, if there is any obstructions in the air intake, make sure the left/right pendulous are not loose and the electrical outlet has power.

Before plugging in the Frame Warmer please make sure, the power switch is off. This can prolong the motor life by avoiding high instantaneous voltage.

When using, don't switch over frequently. Keep the input voltage steady to prolong the life of the power switch and motor.

If the following problems occur: high temperatures, cacophony, noise, rotary speed decreases suddenly, the motor stop working, burning smell, or the outlet is blown. Please **TURN OFF** the machine immediately and check over the machine. Then try re-starting the machine.

Product specifications

Voltage	220V 110V	Heating temperature(Max)	110° C
Frequency	50Hz 60Hz	Noise	≤ 70B(A)
Power	500-650W	Protection Class	II for the Electrical Appliances

Power for heating coil	500-600W	Temperature Adjustment	
Power for Induced Fan	3W		
Air Flow Rate	1-2.0m/s 2-4.0m/s	Weight	0.7kg

Warning

- Don't over heat frames in one spot. Discontinuous using can prolong its service life.
- Don't use the machine if it is not grounded or if it does not have a steady voltage.
- Don't use near water or allow water to get inside the machine. This can cause a short circuit and can cause the motor to burn out.