

DISCLAIMER	1
INTRODUCTION	1
SAFETY INSTRUCTIONS.....	2
OPERATING INSTRUCTIONS.....	2
COMPONENTS.....	3

DISCLAIMER

Amcon: The Eyecare Supply Center makes no representations or warranties, either expressed or implied, with respect to the contents hereof and specifically disclaims any warranties, merchantability or fitness for any particular purpose. Further, Amcon: The Eyecare Supply Center reserves the right to revise this publication and to make changes from time to time in the contents hereof without obligation of this company to notify any person of such revision or changes.

INTRODUCTION

Thank you for purchasing the Amcon Frame Warmer. The Amcon Frame Warmer is ideal for any retail shop or wholesale lab. It offers quick heating with dual temperature switches and provides cold, warm, or hot air. Please read this instruction book before use and keep this manual in a convenient location for future reference.

SAFETY INSTRUCTIONS

To ensure safe use of this product, it is important that you follow the instructions outlined in this manual.

- Please read this user manual before you operate your frame warmer. Save this user manual for future reference.
- When the machine is in use, make sure the airflow is not obstructed so not to damage the heating elements.
- Do not operate or store the frame warmer in a damp area.


OPERATING INSTRUCTIONS

1. Plug unit into 3 prong outlet.
2. Press ON/OFF switch to start cold air flow.
3. Press Temp. 1 switch for warm air flow.
4. Press Temp. 2 switch for hot air flow.
5. Attach frame stand for use when heating frame bridges, frame fronts, and temples.
6. Attach diffuser cap if you prefer heating smaller areas of frames. Be sure to align the hole in the cap with the air outlet.



Contact Amcon immediately if the fan does not activate once the heater is switched on. Please do not touch the temperature switch.

COMPONENTS

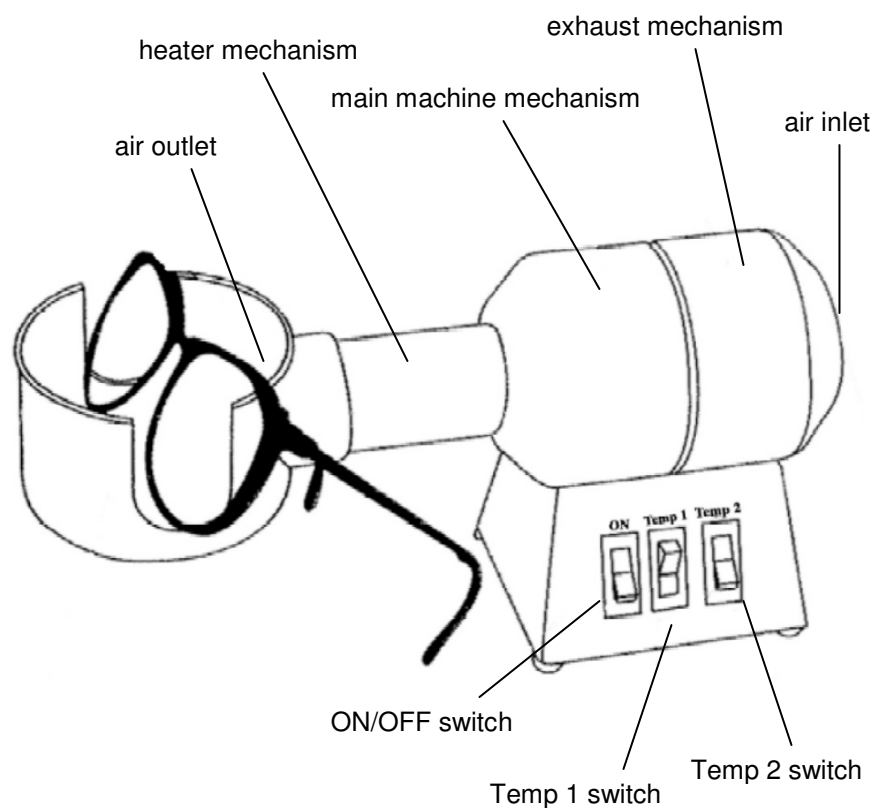

AMCON FRAME WARMER USER'S MANUAL *EQ-3000*

AMCON The Eyecare Supply
CENTER®

1-800-255-6161 • Fax 1-800-397-0013

www.amconlabs.com