

DMV®**ULTRA**

For Hard & Gas Permeable Lenses

REMOVAL - Moisten the suction cup of the DMV® Ultra and guide it to the eye. Be sure the contact lens is centered on the eye before removing it. Align the suction cup so that it is positioned in front of and parallel to the lens. When the suction cup touches the lens, gentle pressure will make the lens adhere to the cup. Merely take the contact lens from the eye. It is safe and easy.

After removal from the eye, carefully slide the lens sideways from the suction cup and it will come off easily.

INSERTION - Place the clean wet lens on the insertion cup at the end of the handle. The lens will adhere in this position while being placed on the eye. Touch the moistened lens to the eye and it will quickly transfer to the eye from the insertion cup.

CLEANING - Each time after using the DMV® Ultra clean it with 70% isopropyl alcohol and allow to air dry, then store the device in its case. Plan to replace it every six months.

PRECAUTION - In the event that the suction cup would be unintentionally attached to the eye, merely slide it to the outside corner and it can be easily removed by turning.

Keep this and all other medical devices away from small children. Use of this contact lens accessory product not recommended with diseased or injured eyes.

Caution: This product contains natural rubber latex which may cause allergic reactions.

"See your eye-care practitioner yearly."

DMV® CORPORATION - ZANESVILLE, OHIO