

PERM-A-DOT™ MARKING SYSTEM

The PERM-A-DOT MARKING SYSTEM is a contact lens marking system which, when properly applied, leaves a permanent semi-indelible mark. No surface etching is required. Burring the surface can leave areas on the lens surface which can cause deposit formation or even bacteria sediment.

PERM-A-DOT is supplied as a complete system, but may be used separately.

How to use the PERM-A-DOT MARKING SYSTEM:

- * Make sure the lens surface is clean and free of oily residue.
- * Shake ink bottle well to eliminate any settlement.
- * Apply small dot with a sharp toothpick or a blunt needle.
- * Allow PERM-A-DOT to dry approximately 60 seconds, times may vary.
- * The developer (remover) is primarily used to wipe any excess ink formation due to oversized dotting. The developer can also be used to thin the PERM-A-DOT in the case of excess drying.

**SPECIAL NOTE: Always recap bottle to prevent evaporation.

PERM-A-DOT is distributed by Amcon. Labs
Call Toll Free at (800)255-616

PERM-A-DOT™

CONTACT LENS MARKING SYSTEM

The perm-A-Dot™ Marking System, when properly applied, leaves a permanent semi-indelible mark. No surface etching is required. Burring the surface leaves areas on the lens surface, which can cause deposit formation and bacteria sediment.

Perm-A-Dot™ is supplied as a complete system but may be used separately.

How to use the Perm-A-Dot™ Marking System:

- Make sure the lens surface is clean and free of oily residue.
- Shake the ink bottle well to eliminate any sediment.
- Apply a small dot with a sharp toothpick or blunt needle.
- Allow Perm-A-Dot™ to dry approximately 60 seconds, times may vary.

The developer (remover) is primarily used to wipe away any excess ink formation due to oversize dotting. The developer can also be used to thin the Perm-A-Dot™ ink in the case of excess drying. If dot does not stick to lens surface, pre-clean the area with developer and reapply.

**** Special Note:** Always recap bottle to prevent evaporation.

Perm-A-Dot™ is harmful if swallowed. DO NOT USE IN EYE

Perm-A-Dot™ is a product of POLYCHEM, INC., USA

Amcon Laboratories, Inc
Phone: 1-800-255-6161
Fax: 1-800-397-0013